
te¢hnol⇔gy for liviæg

Provincial Respiratory Outreach Program • prop@technologyforliving.org • 1.866.326.1245

Technology for Independent Living Program • til@technologyforliving.org • 604.326.0175

Included in this package is the attendant call system that was requested from Technology For Living. To setup the attendant call system, please follow these steps...

1. Plug the switch into the transmitter (small white box)...

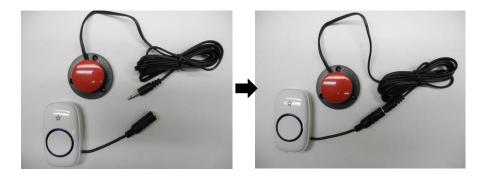

- 2. Place the switch in a location where the user can activate it when they need it.
- 3. Locate the room where you would like the alarm to ring. Plug the receiver (large white box) into an electrical outlet in this room.



- 4. Test the system by having the user activate the switch. The alarm should sound at the receiver.
- 5. If the alarm doesn't sound, please refer to the programming instructions on the back of this page. You may need to program the alarm again.



Provincial Respiratory Outreach Program • prop@technologyforliving.org • 1.866.326.1245

Technology for Independent Living Program • til@technologyforliving.org • 604.326.0175

Troubleshooting tips...

- 1. If the user presses the switch and the blue light on the transmitter (small white box the switch is plugged into) doesn't light up, you will need to replace the A23 battery inside the transmitter.
- 2. If the user presses the switch and the blue light on the transmitter lights up, but the alarm doesn't ring, position the receiver (large white box) closer to the user's location.

Pairing and changing ring tones

Please do the following steps as soon as possible since receiver will exit learning mode automatically if it doesn't get signal from Transmitter in 5 seconds. In order to avoid interference, pair 1 Transmitter only at a time.

- a) Select desired ring tone by pressing Music Control button
- b) Set Receiver into learning mode (pairing mode) by either method i) or method ii) below. A chime and LED flashing will indicate the ready status.
 - i) Press the Volume and Music Control buttons on the Receiver simultaneously.
 - ii) Or press the Volume Control button on the receiver for 5 seconds.
- c) Pair transmitter with Receiver by pressing button on transmitter. You will hear the tune you selected, which means that your operation is completed successfully.

